### ENGLISH SPECIMEN COLLECTION AND HANDLING

Collection of 1 stool sample: follow instructions below.  
Collection of 3 stool samples: Collect the samples from 3 different stools over no more than 3 consequent days.

1. Collect a specimen of stool in a clean dry container or receptacle, for example the Stool Collection Paper provided with the Sample Collection Kit. Use the Stool Collection Paper as follows:
  - Peel off the liner covering the adhesive tape on each end of the collection paper.
  - Lift toilet seat. Unfold collection paper and place on the rim of toilet bowl. Secure the adhesive tabs on the collection paper to the sides of the toilet rim.
  - Make bowel movement onto collection paper.
2. Unscrew (colored cap) and remove the Sample Collection Tube applicator stick. Be careful not to spill or spatter solution from container.
3. Collect random samples by inserting the applicator stick into the stool specimen. Take samples from various surfaces of the stool specimen.
4. Re-insert the applicator stick into the tube and screw the cap tightly. Be careful not to break the tip of the Sample Collection Tube.
5. Write your name and the date of sample collection in the space provided, on the Sample Collection Tube.
6. Return the specimen for testing promptly by mail or in person. The specimen(s) can be stored in the refrigerator (2-8 °C) for no more than 11 days, or at room temperature (max. 25 °C) for no more than 5 days.

**NOTE:** Unless your doctor advises differently, do not collect samples if you have visible blood in your stool or in your urine; during menstruation or 3 days before or after; during constipation; or if you take rectal medication, blood-thinning medication or high doses of NSAIDs (e.g. aspirin, ibuprofen). These factors may cause false positive results.

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