

Biohit Oyj

GASTRIN-17 STABILIZER

Store at 15-25 °C Upon Receipt

INSTRUCTIONS FOR USE



601 050 (1 x 5.5 ml)

601 051 (5 x 5.5 ml)



Biohit Oyj
Laippatie 1
FIN-00880 Helsinki, Finland
Tel: +358-9-773 861
Fax: +358-9-773 2867
E-mail: info@biohit.fi
www.biohithealthcare.com

1. INTENDED USE

The use of Gastrin-17 (G-17) stabilizer is intended to prolong the stability of G-17 in EDTA plasma.

2. PRODUCT CONTENTS AND STABILITY

2.1. Gastrin-17 Stabilizer

Contents: 5.5 ml of the stabilizer in a drop bottle.

Preparation: Ready for use.

Stability: Stable until expiration date when stored at 15-25 °C.

WARNING: The solution can cause irritation!

2.2. Instruction Manual

3. USE OF THE STABILIZER

Use as a 5% solution:

1. Add two drops (50 µl) of the solution into 1 ml of EDTA plasma.
2. Mix by inverting the tube.
3. Treated samples can be stored for 4 days at 20-25 °C. For longer storage the samples should be frozen (preferably at -70 °C, alternatively at -20 °C).

4. PERFORMANCE CHARACTERISTICS

The stabilizer will keep the G-17 concentration in EDTA plasma stable for 4 days at room temperature (20-25 °C). The stabilizer does not interfere with Biohit. Pepsinogen I, Pepsinogen II and *H. Pylori* ELISA kits.

5. DATE OF ISSUE

Gastrin-17 stabilizer

Version 02, 18.12.2012.

6. WARRANTY





The Manufacturer shall remedy all defects discovered in any Product (the "Defective Product") that result from unsuitable materials or negligent

workmanship and which prevent the mechanical functioning or intended use of the Products including, but not limited to, the functions specified in the Manufacturer's specifications for the Products. ANY WARRANTY WILL, HOWEVER, BE DEEMED AS VOID IF FAULT IS FOUND TO HAVE BEEN CAUSED BY MALTREATMENT, MISUSE, ACCIDENTAL DAMAGE, INCORRECT STORAGE OR USE OF THE PRODUCTS FOR OPERATIONS OUTSIDE THEIR SPECIFIED LIMITATIONS OR OUTSIDE THEIR SPECIFICATIONS, CONTRARY TO THE INSTRUCTIONS GIVEN IN THE INSTRUCTION MANUAL.

The period of this warranty for the Distributor is defined in the instruction manual of the Products and will commence from the date the relevant Product is shipped by the Manufacturer. In case of interpretation disputes the English text applies.

This Biohit diagnostic kit has been manufactured according to our ISO 9001 / ISO 13485 quality management protocols and has passed all relevant Quality Assurance procedures related to this product.

7. EXPLANATION OF THE SYMBOLS USED IN LABELLING

	Catalogue number
	Batch code
	Use by
 +15...+25°C	Temperature limitation Store at +15...+25 °C

8. ORDERING INFORMATION

Gastrin-17 Stabilizer

Cat. No. 601 050, 1 x 5.5 ml

Cat. No. 601 051, 5 x 5.5 ml

Headquarters

BIOHIT OYJ

Laippatie 1

00880 Helsinki, Finland

Tel: +358-9-773 861

Fax: +358-9-773 2867

E-mail: info@biohit.fi

www.biohithealthcare.com