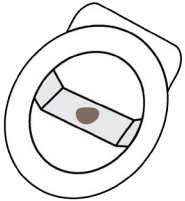
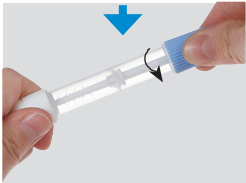
## BIOHIT COLONVIEW quick test Sample Collection Kit

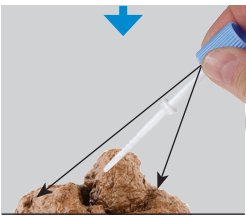
1.

2.


3.

4.

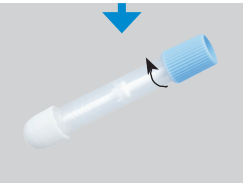5.

ENGLISH

### SPECIMEN COLLECTION AND HANDLING

Collection of 1 stool sample: follow instructions below. Collection of 3 stool samples: Collect the samples from 3 different stools over no more than 3 days.

Collect a specimen of stool in a clean dry container, for example the Stool Collection Paper provided with the Sample Collection Kit. Use the Stool Collection Paper as follows: Peel off the liner covering the adhesive tape on each end of the collection paper. Lift toilet seat. Unfold collection paper and place the adhesive tape carefully on the rim of toilet bowl.

1. Make bowel movement onto collection paper.
2. Unscrew the blue end and remove the Sample Collection Tube applicator stick. Be careful not to spill or spatter solution from container.
3. Collect a random sample by poking the applicator stick into three different places of the stool. Only the small grooves in the applicator stick need to be filled. See as an example the picture 4 of the amount in the stick. Please note that too much stool can cause invalid result. A small sample is enough, **DO NOT OVERLOAD THE TUBE.**
4. Re-insert the applicator stick into the tube and screw the cap tightly
5. If you are using the Stool Collection Paper, release adhesive tapes and discard the paper into appropriate waste basket.
6. Write patient name and sample collection date in the space provided on the Sample Collection Tube.
7. Store the samples in a refrigerator inside the provided plastic bag until the third sample is collected. Return the samples for testing without delay by mail or in person. Please note that the samples can be stored in a refrigerator (2-8 °C) for no longer than 11 days, including the laboratory testing. The samples can only be stored in room temperature temporarily.

**NOTE:** Unless your doctor advises differently, do not collect samples if you have visible blood in your stool or in your urine; during menstruation or 3 days before or after; during constipation; or if you take rectal medication, blood-thinning medication or high doses of NSAIDs (e.g. aspirin, ibuprofen). These factors may cause false positive results.

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