

# BIOHIT LACTOSE INTOLERANCE

quick test

## QUICK GUIDE

- Allow the test components to reach room temperature before use.
- Biopsy specimen(s) is recommended to be taken from the postbulbar duodenum.
- If there is blood on the specimen, dab the biopsy briefly on a sterile gauze pad.
- Mix all reagents before use by turning the bottles upside down for a few times and tap the bottles on the table to ensure the liquid returns to the bottle.

1. Place the biopsy specimen into the well.


Add two drops of **reagent 1**.

Close the cover of the well and shake the plate 5-6 times horizontally on the table.

Incubate for 15 minutes.



15 min

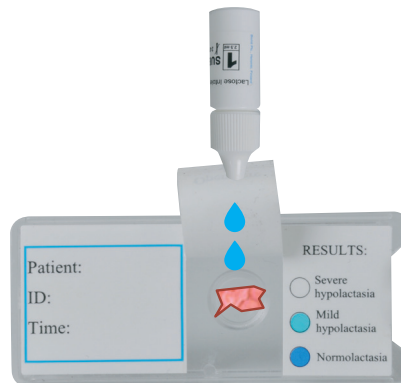


2. Add one drop of **reagent 2** and two drops of **reagent 3**.

Close the cover of the well and shake the plate 5-6 times horizontally on the table.

Incubate for 5 minutes.



5 min

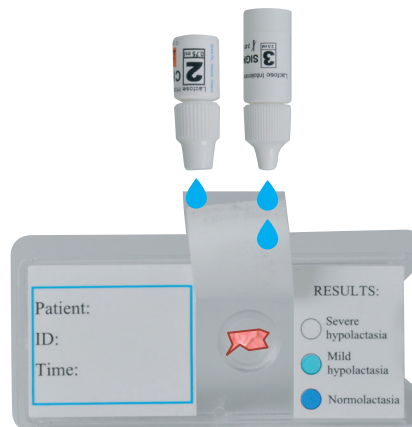

3. Read the results against the color scale on the plate or use the interpretation chart included in the product package.



**BIOHIT HealthCare**

Innovating for Health

BIOHIT Oyj

Laippatie 1, 00880 Helsinki, Finland

Tel: +358 9 773 861, E-mail: [info@biohit.fi](mailto:info@biohit.fi)

[www.biohithealthcare.com](http://www.biohithealthcare.com)