

SAMPLE STABILITY

19.2.2016

Analyte	Sample	RT	+4°	-20°	-70°
Calprotectin	fecal	4 days	4 days	2 years	2 years
Celiac QT	finger prick/ whole blood NB:GastroPanel sample with stabilizer cannot be used	recommended immediate use	N.A	N.A	N.A
Celiac QT	plasma NB:GastroPanel sample with stabilizer	recommended immediate use	3 days	for longer storage	N.A
Celiac QT	serum NB:GastroPanel sample with stabilizer cannot be used	recommended immediate use	2 days	for longer storage	N.A
ColonView/ Hb, Hb/Hp complex	fecal in ColonView sample tube	5 days	11 days	do not freeze	do not freeze
Cortisol	plasma without stabilizer	not known	5 days	6 months	6 months
GastroPanel	plasma with stabilizer	3 days	7 days	2 weeks	2 years
Vitamin B12	serum	<16 hours	4 weeks	6 months	6 months
Vitamin D	serum	< 24 hours	3 days	1 month	2 years

LOM-9B-04-02