

## GASTROPANEL EXAMINATION - INSTRUCTIONS FOR THE PATIENT

You have been scheduled for GastroPanel examination.

Please notice the following before the blood sample collection:

- Avoid eating, drinking and smoking at least 4 hours before sample collection.
- The patient is allowed to take prescribed, regular medication. However, kindly report of the use of PPI-medicines to the person who is taking your blood sample, as some medicines may interfere with the secretion of gastric fluids. See the list of PPI-medicines which inhibit the secretion of gastric acid:

### PROTON-PUMP INHIBITORS (PPI-MEDICINES):

**Esomeprazole;** ESOMEPRAZOL ORION, ESOMEPRAZOL RATIOPHARM, ESOMEPRAZOL SANDOZ, NEXIUM, NEXIUM 10 mg

**Lansoprazole;** GASTERIX, LANSOPRAZOL ACTAVIS 15 mg, ZOLT

**Omeprazole;** LOSEC, LOSEC MUPS, OMEPRAZOL ACTAVIS, OMEPRAZOL PENZA, OMEPRAZOL RATIOPHARM, OMEPRAZOL SANDOZ, OMESTAD

**Pantoprazole;** PANTOPRAZOL ACTAVIS, PANTOPRAZOL PENZA , PANTOPRAZOL SANDOZ, PANZOR, SOMAC CONTROL, SOMAC

**Rabeprazole;** PARIET

Best regards  
Biohit Laboratory Services  
Laippatie 1  
00880 Helsinki  
Finland  
Tel. +358-400 603 222

04022016