

BIOHIT HELICOBACTER PYLORI UFT300

quick test

BIOHIT HealthCare

Innovating for Health



Product specifications

- Sample: gastric biopsy specimen
- Test principle: *H. pylori* urease activity
- Kit components: test tube/plate and reagent, instruction for use
- Storage: +18-25°C
- Quality control: positive control available

Ordering details:

BIOHIT <i>Helicobacter pylori</i> UFT300 quick test with plate	
Cat. No.	Item
602005PLA	BIOHIT <i>Helicobacter pylori</i> UFT300, 5 tests
602019PLA	BIOHIT <i>Helicobacter pylori</i> UFT300, 50 tests

BIOHIT <i>Helicobacter pylori</i> UFT300 quick test with tube	
Cat. No.	Item
602019	BIOHIT <i>Helicobacter pylori</i> UFT300, 50 tests
602021	BIOHIT <i>Helicobacter pylori</i> UFT300, 100 tests

Cat. No.	Item
602017	Positive control reagent



BIOHIT HELICOBACTER PYLORI UFT300

quick tests

ULTRA-FAST, READY-TO-USE TEST
PLATE AND TUBE VERSIONS



Biohit Oyj

Laippatie 1
00880 Helsinki
Finland
Tel. +358 9 773 861
info@biohit.fi

Biohit HealthCare Srl

Via Boncompagni, 3
20139 Milano
Italy
Tel +39 02 38238113
info.italy@biohit.fi

Biohit HealthCare Ltd.

Pioneer House, North Rd
Ellesmere Port, CH65 1AD,
United Kingdom
Tel. +44 151 550 4 550
info@biohithealthcare.co.uk

Order brochures: info@biohit.fi

www.biohithealthcare.com

BIOHIT HealthCare

Innovating for Health

490093EN-04 10/2020

In the USA and Japan for research use only.

Reference:

1. Vaira D, Vakil N, Gatta L, Ricci C, Perna F, Saracino I, Fiorini G, Holton J. Accuracy of a new ultrafast rapid urease test to diagnose *Helicobacter pylori* infection in 1000 consecutive dyspeptic patients. *Aliment Pharmacol Ther* 31 (2010), 331-338.



5 mins (max.)
FROM BIOPSY

BIOHIT HELICOBACTER PYLORI UFT300

quick test

Simple test procedure with plate

1. Place 1 or more biopsies into the well of the plate



2. Add one drop of the reagent solution. Cover the plate with label and shake for 5 seconds



3. Read after 5 minutes.

Negative



Positive



Ultra-fast *H. pylori* detection from a biopsy

The BIOHIT *H. pylori* UFT300 is a true quick test for the detection of *H. pylori* from a biopsy. The biopsy taken during gastroscopy can be tested immediately to diagnose *H. pylori* infection or to check the success of eradication therapy. The result is ready in only 5 minutes enabling diagnosis and reporting during the same visit to the doctor. The UFT300 quick test has excellent sensitivity and specificity which makes it a highly reliable tool for diagnostics.

- Ready-to-use test kit
- Results ready in 5 min
- Storage in room temperature
- Sensitivity 94,5 %, Specificity 100 %¹
- Testing and reporting during one appointment

Biopsy testing could not be easier

The BIOHIT *H. pylori* UFT300 quick test is easy and effortless to use. The biopsy specimen is placed into the test tube/well of the plate and mixed with the test reagent. Color change indicates the presence of *H. pylori* in the specimen. The procedure is safe and hygienic for the user, and the interpretation of the result doesn't require specialist training.

Simple test procedure with tube

1. Place 1 or more biopsies into the tube



2. Add one drop of the reagent solution. Cap the tube and shake for 5 seconds



3. Read after 5 minutes.



Negative



Positive

