SUOMI

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Acetium® capsule

Intended use

The product protects the gastric mucosa from acetaldehyde in people who have an acid-free stomach. An acid-free stomach may be due to atrophic gastritis of the stomach mucosa, which is usually caused by a chronic *Helicobacter pylori* infection. An acid-free stomach may also be caused by long-term use of acid-suppression medicines such as H₂-receptor blockers (H₂-blockers) or proton pump inhibitors (PPI). An acidity of the stomach enables oral microbes to grow and produce acetaldehyde in the stomach. Acetium capsules slowly release L-cysteine, an amino acid that binds acetaldehyde derived from alcohol, food and microbes in the stomach. The World Health Organization (WHO) has classified acetaldehyde contained in alcoholic beverages and endogenously produced from alcohol as a carcinogenic substance.

Directions for use

Take 1-2 capsules with water along with a meal or alcoholic beverages. If drinking continues, you may take 1-2 additional capsules. The capsule needs to be swallowed as whole. The recommended maximum daily dose is 10 capsules.

Storage and shelf life

Keep out of view and reach of children. Store in a dry place at room temperature below 25 °C. The product must not be used after the expiry date indicated on the package.

Contraindications

The product is not suitable for children, during pregnancy or breastfeeding, or for persons suffering from renal dysfunction or cystinuria. Consult a health care practitioner prior to use if you are,

- using penicillin antibiotics. Consumed concurrently, Acetium may reduce the efficacy of penicillin antibiotics. Acetium capsules should be ingested 2-3 hours before the intake of penicillin antibiotics.
- using nitroglycerin medicines. Acetium may increase the effect of nitroglycerin medicines. High doses of Acetium (more than two capsules, three times per day) in combination with nitroglycerin medicines may cause a headache.
- using Angiotensin-converting enzyme (ACE) inhibitors (commonly used for treatment of hypertension). Acetium may potentiate the antihypertensive effects of ACE inhibitors.

Do not take this product if you are allergic to any of the ingredients. If you notice adverse effects, contact a doctor or the manufacturer.

A single oral dose of over 5 grams of L-cysteine may cause nausea and dizziness. If the symptoms are severe or prolonged, contact your doctor.

Composition

Hard white hypromellose capsule containing white granulated powder. Each capsule contains 100 mg L-cysteine. Other ingredients: Eudragit® RS-PO, hypromellose, calcium hydrogen phosphate, titanium dioxide. Sugar-free, lactose-free and gluten-free. Does not contain human or animal derived ingredients.

Package form and size

60 capsules in a blister pack (PVC/aluminum) inside a carton.

Instructions for disposal

The carton is recyclable. The foil package and any remaining capsules may be disposed of as mixed waste. Discard the used package and remaining capsules according to local and national regulations.

Biohit Oyj
Laippatie 1
FI-00880 Helsinki
Finland

	Single use, do not re-use
	Use by
	Lot number
	Consult instructions for use
	Catalogue number
	Manufacturer

This package insert was last revised
2.10.2020