

BIOHIT HealthCare

Innovating for Health

PEPSINOGEN II

ELISA kit for the measurement of human pepsinogen II in EDTA plasma and serum

Instructions for use

REF 601020.02

IVD



For in vitro diagnostic use
Store at 2-8 °C Upon Receipt

Biohit Oyj Laippatie 1, FI-00880 Helsinki, Finland

Tel. +358 9 773 861, info@biohit.fi, www.biohithealthcare.com

CONTENTS

- 1. INTENDED USE.....4
- 2. CLINICAL BACKGROUND.....4
- 3. PRINCIPLE OF THE TEST4
- 4. WARNINGS AND PRECAUTIONS5
- 5. SPECIMEN COLLECTION AND HANDLING.....5
- 6. KIT CONTENTS, REAGENT PREPARATION AND STABILITY FOR MATERIALS PROVIDED.....6
 - 6.1. Microplate6
 - 6.2. Washing Buffer Concentrate (10 x)6
 - 6.3. Diluent Buffer6
 - 6.4. Blank Solution.....6
 - 6.5. Calibrators7
 - 6.6. Control7
 - 6.7. Conjugate Solution7
 - 6.8. Substrate Solution7
 - 6.9. Stop Solution7
 - 6.10. Incubation Covers7
 - 6.11. Instructions for Use7
- 7. MATERIALS REQUIRED BUT NOT PROVIDED8
- 8. STORAGE AND STABILITY8
- 9. TEST PROCEDURE9
- 10. RESULTS10
 - 10.1. Quality Control Values10
 - 10.2. Calculation of the Results10
 - 10.3. Interpretation of the Results11
- 11. LIMITATIONS OF THE PROCEDURE.....12
- 12. PERFORMANCE CHARACTERISTICS.....12
- 13. REFERENCES.....16
- 14. DATE OF ISSUE17
- 15. WARRANTY18
- 16. ORDERING INFORMATION18
- 17. EXPLANATION OF THE SYMBOLS USED IN LABELS.....19
- 18. SHORT OUTLINE OF THE PROCEDURE20

APPENDIX: QUALITY CONTROL CERTIFICATE

1. INTENDED USE

Pepsinogen II (PGII) test is an *in vitro* microplate-based quantitative enzyme-linked immunosorbent assay (ELISA) for the determination of human pepsinogen II from EDTA plasma or serum samples. The test aids in diagnosis of atrophic gastritis in the gastric corpus from the patients at risk to develop malignant cellular changes in stomach mucosa, with the Pepsinogen I test (REF 601010.01) by a PGI/PGII ratio. Furthermore, PGII is used for screening of mucosal inflammation.

The test can be conducted either manually or automatically and is to be used by healthcare professionals

2. CLINICAL BACKGROUND

Pepsinogen II is produced by chief cells and mucous neck cells of the gastric mucosa, in pyloric glands in the gastric antrum, and Brunner's glands in the proximal duodenum. The ratio of concentration of Pepsinogen I (PGI) to PGII in plasma or serum of normal subjects is about 4:1 (1).

The PGI/PGII ratio decreases linearly with increasing grade of atrophic gastritis in the corpus (2, 3). The ratio is < 3.0 when atrophic gastritis is advanced (moderate or severe) in the gastric corpus (3). It has been shown that the risk of gastric cancer is increased (5-fold) when the PGI/PGII ratio is low (5-14). This test is intended as an additional tool in the diagnosis of atrophic corpus gastritis, which is also a risk state for gastric cancer (2, 4). The Pepsinogen II assay is used together with the Pepsinogen I assay by which the PGI/PGII ratio is determined. As will as with Gastrin-17 confirms atrophic corpus gastritis.

3. PRINCIPLE OF THE TEST

This PGII ELISA is based on a sandwich enzyme immunoassay technique with a PGII specific capture antibody adsorbed on a microwell plate and detection antibody labeled with horseradish peroxidase (HRP). The assay proceeds according to the following reactions:

1. Human PGII specific monoclonal antibodies attached to polystyrene surface bind PGII molecules present in the sample.
2. Wells are washed to remove residual sample.
3. HRP-conjugated detection antibodies are added into the wells and they bind to PGII molecules.
4. The wells are washed and TMB-substrate is added. The substrate is

oxidized by the enzyme and a blue colored end product is produced.

5. The enzyme reaction is terminated with stop solution. The solution in the microwells should turn yellow. The intensity of the yellowish color developed is directly related to the PGII concentration of the sample.

4. WARNINGS AND PRECAUTIONS

For *in vitro* diagnostic use

CAUTION: Handle plasma and serum samples as potentially biohazardous material.

All samples should be regarded as potentially contaminated and treated as if they were infectious. Please refer to the U.S. Department of Health and Human Services (Bethesda, MD., USA) publication Biosafety in Microbiological and Biomedical Laboratories, 2009, 5th ed. (CDC/NIH) and No. (CDC) 21-1112 on reports of laboratory safety procedures on different diseases or any other local or national regulation.

Always use protective gloves when handling patient samples. Use a safety pipetting device for all pipetting. Never pipette by mouth. Read all instructions prior to performing this assay. All reagents provided in the kit can be disposed by pouring into a sink and flushing with an excess of tap water. Dispose other materials according to local waste management legislation.

Components containing ProClin may cause an allergic skin reaction (see Safety Data Sheet). Dispose of solutions containing ProClin according to local waste management legislation

5. SPECIMEN COLLECTION AND HANDLING

Fasting for 4 hours is recommended prior to blood sampling. The blood sample is collected by venipuncture into e.g. a plastic EDTA or serum tube without additives. Plasma blood tubes are mixed immediately by turning them upside down 5-6 times, and tubes for serum are allowed to clot (for minimum 30 minutes) at room temperature (20-25°C). Serum after clotting, and plasma immediately, is separated by centrifugation (e.g. plastic tube, relative centrifugal force up to 2000 G, 10-15 minutes).

Samples can be stored for 7 days in a refrigerator at 2..8°C, and 3 days at room temperature. For longer storage the samples should be stored frozen (preferably at -70°C, alternatively at -20°C). Mix the samples thor-

oughly after thawing. Avoid repeated freezing and thawing of the samples. Grossly hemolysed, lipemic or turbid specimens should be avoided.

Please refer to Gastrin-17 Advanced ELISA instructions for use if testing the Biohit GastroPanel ELISA assays (Pepsinogen I, Pepsinogen II, Gastrin-17 Advanced, *Helicobacter pylori* IgG antibodies) from the same sample.

6. KIT CONTENTS, REAGENT PREPARATION AND STABILITY FOR MATERIALS PROVIDED

The reagents are sufficient for 96 wells and three separate runs. Reagents of different kit lots should not be mixed.

6.1. Microplate

Contents: 12 x 8 strips in frame coated with high-affinity, monoclonal anti-human-PGII IgG₁

Preparation: Ready for use. Do not combine strips/wells from different microplates, even if they would have the same lot number.

Stability: Stable until expiry date. Discard the strips after use. Note that after opening the microplate foil, some crystals may be formed in the bottom of the wells and may affect the results. Discard the crystalized strips. Do not combine strips/wells from different microplates, even if they would have the same lot number

6.2. Washing Buffer Concentrate (10 x)

Contents: 120 ml of 10 x phosphate buffer concentrate containing Tween 20 and 0.1% ProClin 300 as preservative.

Preparation: Dilute 1 to 10 (e.g. 100 ml + 900 ml) with distilled water and mix well.

Stability: The diluted solution is stable for two weeks refrigerated (2-8 °C).

6.3. Diluent Buffer

Contents: 100 ml of phosphate buffer containing casein, Tween 20, 0.1% ProClin 300 as preservative and red dye extract.

Preparation: Ready for use.

Stability: Stable until expiry date.

6.4. Blank Solution

Contents: One vial containing 1.5 ml of human serum-based phosphate buffer with 0.1% ProClin 300 as preservative.

Preparation: Ready for use.

Stability: Stable until expiry date.

6.5. Calibrators

Contents: Three vials each containing 1.5 ml of human serum-based calibrators with 0.1% ProClin 300 as preservative. The calibrators have lot-specific PGII values of approximately 6.3, 12.5 and 50 µg/l. The exact PGII concentration of the calibrators is labeled on the vials.

Preparation: Ready for use.

Stability: Stable until expiry date.

6.6. Control

Contents: One vial containing 1.5 ml of human serum-based PGII control with 0.1% ProClin 300 as preservative. The expected PGII level of the control serum is indicated on the label of the vial.

Preparation: Ready for use.

Stability: Stable until expiry date.

6.7. Conjugate Solution

Contents: 15 ml of HRP-conjugated anti-human-PGII in stabilizing buffer with 0.1% ProClin 300 as preservative.

Preparation: Ready for use.

Stability: Stable until expiry date.

6.8. Substrate Solution

Contents: 15 ml of tetramethylbenzidine (TMB) in aqueous solution.

Preparation: Ready for use.

Stability: Stable until expiry date. Avoid direct exposure to light.

6.9. Stop Solution

Contents: 15 ml of 0.1mol/l sulphuric acid.

Preparation: Ready for use.

Stability: Stable until expiry date.

6.10. Incubation Covers

Three plastic sheets to cover the microplate during incubation.

6.11. Instructions for Use

7. MATERIALS REQUIRED BUT NOT PROVIDED

- ▶ Distilled or deionized water
- ▶ Micropipettes and disposable tips, to accurately deliver 20 - 1000 μ l
- ▶ Pipettes to accurately deliver 1-10 ml
- ▶ 8-channel pipette delivering 100 μ l
- ▶ Graduated cylinder, 1000 ml
- ▶ Vortex mixer for sample dilutions
- ▶ Test tubes for specimen dilutions
- ▶ Microplate washer
- ▶ Paper towels or absorbent paper
- ▶ Timer
- ▶ Incubator, 37°C
- ▶ Microplate reader, 450 nm
- ▶ E. g. plastic blood collection tube for plasma or serum
- ▶ Container for ice-water bath

8. STORAGE AND STABILITY

Store the Pepsinogen II kit refrigerated (2-8°C). When stored at these temperatures, the kit is stable until the expiration date printed on the box label and the label of each individual kit component. Do not freeze or expose the kit to high temperatures, or store at above 8°C when not in use. The substrate solution is sensitive to light. The microplate or individual strips should not be removed from the foil pouch until equilibrated to room temperature (20-25°C). Unused strips must be returned to the foil pouch, sealed and stored at 2-8°C.

Do not use any of the reagents after the expiration date printed on the label. Do not use reagents from kits with different lot numbers or substitute reagents from kits of other manufacturers. Use only distilled or deionized water. The components of the kit are provided at precise concentrations. Further dilution or other alterations of the reagents may cause incorrect results.

Indication of Kit Deterioration

Liquid components should not be visibly cloudy or contain precipitated material. At 2-8°C, the washing buffer concentrate may, however, partially crystallize, but the crystals will dissolve by mixing at room temperature (20-25°C). The substrate solution should be colorless or pale blue/yellow. Any other color indicates deterioration of the substrate solution.

9. TEST PROCEDURE

PRELIMINARY PREPARATIONS

Allow all reagents and the microplate to reach room temperature (e.g. 20-25°C). Dilute the washing buffer concentrate 1 to 10 (e.g. 100 ml + 900 ml) with distilled or deionized water. Frozen samples should be thawed fast in a room temperature water bath with occasional mixing. Once they are almost thawed, place them in a crushed ice bath. **Read the complete assay procedure before starting. It is recommended that the calibrators and samples are applied on the plate as duplicates. It is necessary to use calibrators and the control in each test run.**

Mix all the reagents and samples well before use.

STEP 1: SPECIMEN DILUTION

Dilute the mixed plasma or serum samples 1 to 5 (e.g. 100 μ l + 400 μ l) with the diluent buffer, **mix well**. Return the rest of the sample back to the freezer.

STEP 2: SAMPLE

Mix and pipette 100 μ l of the blank solution (BS), the calibrators (CAL1-CAL3), the control and diluted samples (S1, S2 etc.) into the wells as duplicates (see Figure 1). Cover the plate with the incubation cover. Incubate 60 minutes at room temperature (20-25°C).

	1	2	3	4
A	BS	BS		
B	CAL1	CAL1		
C	CAL2	CAL2		
D	CAL3	CAL3		
E	Control	Control		
F	S1	S1		
G	S2	S2		
H	etc.	etc.		

Figure 1. Pipetting Order.

STEP 3: WASHING

Wash the wells three times with 350 μ l of the diluted (1 to 10) washing buffer and gently tap the inverted plate a few times on a clean paper

towel.

STEP 4: CONJUGATE

Pipette 100 µl of the mixed conjugate solution into the wells, preferably with an 8-channel pipette. Cover the plate with the incubation cover. Incubate for 60 minutes at room temperature (20-25°C).

STEP 5: WASHING

Wash the wells three times with 350 µl of the diluted (1 to 10) washing buffer and gently tap the inverted plate a few times on a clean paper towel.

STEP 6: SUBSTRATE

Pipette 100 µl of the mixed substrate solution into the wells with an 8-channel pipette. Start the incubation time after pipetting the substrate solution into the first strip and continue the incubation for 30 minutes at room temperature (20-25°C). Avoid direct exposure to light during incubation.

STEP 7: REACTION STOP

Pipette 100 µl of the mixed stop solution with an 8-channel pipette into the wells.

STEP 8: MEASURING OF RESULTS

Measure the absorbance at 450 nm within 30 minutes.

10. RESULTS

10.1. Quality Control Values

Good Laboratory Practice requires the use of appropriate controls to establish that all the reagents and protocols are performing as designated. The Pepsinogen II ELISA is provided with control serum. Quality control charts should be maintained to follow the performance of the control, or appropriate statistical methods should be used to analyze control values, which should fall within the appropriate confidence intervals employed in each laboratory.

10.2. Calculation of the Results

Assay results can be analyzed by using a) manual method or b) automated methods, where the absorbance readings are converted to pepsinogen II concentrations. Since the calibrators are ready to use, the results of the patient samples are not multiplied by the dilution factor.

a) Manual Method

Calculate the mean absorbance of the duplicate determinations of the blank solution, the calibrators, the control and samples. Subtract the mean of the blank solution from itself (consider this as the first point of the calibration curve), the calibrators, the control and the samples. Graph the calibration curve by plotting the mean absorbance for the first point and each calibrator (y-axis) against the PGII concentrations given for the calibrators (x-axis). Draw a best-fit curve to construct a calibration curve. Use the mean absorbance value for each sample and the control to interpolate the PGII value from the calibration curve.

b) Automated Methods

There are several computer programs available for interpolating the unknown concentrations automatically. A simple 2nd order polynomial fit is adequate for interpolating unknown concentrations within the calibrator range. However, if sample absorbance values exceeds the absorbance value of the highest calibrator, a more complex extrapolating algorithm may be more appropriate. A typical calibration curve is shown in Figure 2.

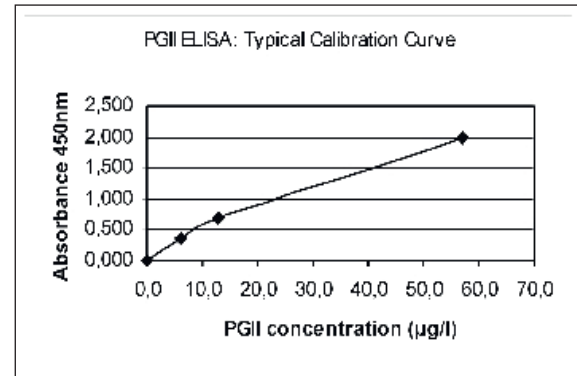


Figure 2. Example of a Typical Calibration Curve.

10.3. Interpretation of the Results

A PGI/PGII ratio lower than 3.0 indicates that the patient has advanced corpus atrophy and an increased risk of gastric cancer. It is recommended that the given limits are considered as guidelines. Also the PGII results

determined for a given specimen with assays from different manufacturers can vary due to differences in standardization, assay methods and reagent specificity. Results obtained by different manufacturers' assay methods should not be used interchangeably.

11. LIMITATIONS OF THE PROCEDURE

As with any diagnostic procedure, Biohit Pepsinogen II ELISA results must be interpreted together with the patient's clinical presentation and any other information available to the physician. The NSAIDs may increase the PGII levels by mucous membrane irritation.

12. PERFORMANCE CHARACTERISTICS

Within-Assay Imprecision

The within-assay imprecision was determined with four serum samples. These samples were run as 18 replicates in one run.

Sample	Mean PGII (µg/l)	CV%
1	4.6	3.2
2	8.8	5.6
3	13.4	5.1
4	21.7	4.0

Between-Assay Imprecision

The between-assay imprecision was evaluated in six assays using four serum samples. The PGII concentration of these samples was measured as duplicates.

Sample	Mean PGII (µg/l)	CV%
1	3.3	2.8
2	6.7	3.4
3	16.3	2.3
4	16.7	3.4

Specificity/Cross-Reactivity

The cross-reaction and interference by PGI was tested by spiking three serum samples with PGI (rPGI, company X) at concentrations up to 500

µg/l. The test showed no significant increase or reduction in the signal of samples with a PGI concentration of up to 500 µg/l.

Sample	PGI added (µg/l)	Observed (µg/l)	Added/0-Sample (%)
1	-	30.5	-
	50	30.2	99
	100	31.5	103
	500	30.2	99
2	-	12.3	-
	50	12.8	104
	100	12.5	102
	500	12.8	104
3	-	5.7	-
	50	5.9	104
	100	5.8	102
	500	6.0	105

Analytical Sensitivity

The sensitivity of the PGII ELISA was determined according to the NCCLS (CLIA) guideline Consensus Standards for Medical Testing (NCCLS EP17-A).

The limit of blank (LoB) was determined as the 95th percentile under which the measurement results of 60 blank samples (blank solution) fall. The LoB was found to be 0.6 µg/l.

The limit of detection (LoD), which is the lowest actual amount of analyte that may be detected with 95% confidence, was assessed using four EDTA plasma and serum samples that contain PGII levels close to the observed LoB value. The LoD was found to be 0.7 µg/l for EDTA plasma and 0.8 µg/l for serum samples.

The limit of quantitation (LoQ), i.e. the lowest amount of analyte in a sample that can be reliably detected, was calculated assuming a coefficient of variation (CV%) of 20% at the level of LoD. Accordingly, the LoQ was calculated to be $1.2 \times \text{LoD} = 0.8 \mu\text{g/l}$ for EDTA plasma and $1.0 \mu\text{g/l}$ for serum samples.

Recovery

Five serum samples were spiked with 10, 20 and 40 µg/l of human pep-

sinogen II (purified human PGII, Biohit Diagnostics).


The average recovery was:

10 µg/l 95.1%
 20 µg/l 96.3%
 40 µg/l 98.9%

Correlation

Correlation was shown with the relationship between serum levels of pepsinogen II and histological status.

Ratio of Pepsinogen I to pepsinogen II – atrophic corpus gastritis – a population sample (Borch et al, University Hospital of Linköping, Sweden).

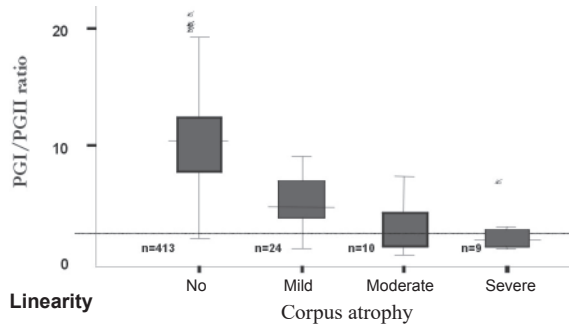

Linearity

Four serum samples were assayed in serial dilutions with the diluent buffer to determine the linearity of Biohit Pepsinogen II ELISA. Results are listed in the table below.

Sample	Dilution Factor	Observed (µg/l)	Expected (µg/l)	Recovery (%)
1	1	23.8	-	-
	2	12.8	11.9	108
	4	6.4	6.0	107
2	1	9.4	-	-
	2	5.1	4.7	109
	4	2.6	2.3	113
3	1	11.0	-	-
	2	5.5	5.5	100
	4	3.0	2.8	107
4	1	28.6	-	-
	2	15.7	14.3	110
	4	7.9	7.1	111

13. REFERENCES

1. Samloff IM. Peptic activity and peptic inhibitors. In: Gitnick G, ed. Principles and practice of gastroenterology and hepatology. New York: Elsevier, 1988; 154-163.
2. Varis K. Surveillance of pernicious anemia. In Precancerous Lesions of the Gastrointestinal Tract. Scherlock P, Morson PC, Barbara L, Veronesi U (eds), Raven Press, New York, 1983;189-194.
3. Varis K, Samloff IM, Ihamäki T, Siurala M. An appraisal of tests for severe atrophic gastritis in relatives of patients with pernicious anemia. *Dig Dis Sci* 1979; 24:187-191.
4. Sipponen P, Kekki M, Haapakoski J, Ihamäki T, Siurala M. Gastric cancer risk in chronic atrophic gastritis: statistical calculations of cross sectional data. *Int J Cancer* 1985; 35:173-177.
5. Miki K, Ichinose M, Ishikawa KB, Yahagi N, Matsushima M, Kakei N, Tsukada S, Kido M, Ishihama S, Shimizu Y, Suzuki T, Kurokawa K. Clinical application of serum pepsinogen I and II levels for mass screening to detect gastric cancer. *Jpn J Cancer Res* 1993; 84:1086-1090.
6. Hattori Y, Tashiro H, Kawamoto T, Kodama Y. Sensitivity and specificity of mass screening for gastric cancer using the measurement of serum pepsinogens. *Jpn J Cancer Res* 1995; 86:1210-1215.
7. Yoshihara M, Sumii K, Haruma K, Kiyohira K, Hattori N, Tanka S, Kajiyama G, Shigenobu T. The usefulness of gastric mass screening using serum pepsinogen levels compared with photofluorography. *Hiroshima J Med Sci* 1997; 46:81-86.
8. Miki K, Ichinose M, Shimizu A, Huang SC, Oka H, Furihata C, Matsushima T, Takahashi K. Serum pepsinogens as a screening test of extensive chronic gastritis. *Gastroenterol Jpn* 1987; 22:133-141.
9. Kodoi A, Yoshihara M, Sumii K, Haruma K, Kajiyama G. Serum pepsinogen in screening for gastric cancer. *J Gastroenterol* 1995; 30: 452-460.
10. Aoki K, Misumi J, Kimura T, Zhao W, Xie T. Evaluation of cut-off levels for screening of gastric cancer using serum pepsinogens and distribution of levels of serum pepsinogen I, II and of PG1/PGII ratios in a gastric cancer case-control study. *J Epidemiol* 1997; 7:143-151.
11. Kikuchi S, Wada O, Miki K, Nakajima T, Nishi T, Kobayashi O, Inaba Y. Serum pepsinogen as a new marker for gastric carcinoma among young adults. Research group on prevention of gastric carcinoma among young adults. *Cancer* 1994; 73:2695-2702.
12. Farinati F, Di Mario F, Plebani M, Cielo R, Fanton MC, Valiante F, Masiero M, DeBoni M, Della Libera G, Burlin A. Pepsinogen A/Pepsinogen C or Pepsinogen A multiplied by gastrin in the diagnosis of gastric cancer. *Ital J Gastroenterol* 1991; 23:194-206.
13. Borch K, Axelsson CK, Halgreen H, Damkjaer Nielsen MD, Ledin T, Szesci PB. The ratio of pepsinogen A to pepsinogen C: a sensitive test for atrophic gastritis. *Scand J Gastroenterol* 1989; 24:870-876.
14. Yoshihara M, Sumii K, Haruma K, Kiyohira K, Hattori N, Kitadai Y, Komoto K, Tanka S, Kajiyama G. Correlation of ratio of serum pepsinogen I and II with prevalence of gastric cancer and adenoma in Japanese subjects. *Am J Gastroenterol* 1998; 93:1090-1096.
15. Agreus L, Kuipers EJ, Kupcinskas L, et al Rationale in diagnosis and screening of atrophic gastritis with stomach-specific plasma biomarkers.

14. DATE OF ISSUE

Pepsinogen II kit insert.
Version 09, 05-2022

15. WARRANTY

The Manufacturer shall remedy all defects discovered in any Product (the "Defective Product") that result from unsuitable materials or negligent workmanship and which prevent the mechanical functioning or intended use of the Products including, but not limited to, the functions specified in the Manufacturer's specifications for the Products. ANY WARRANTY WILL, HOWEVER, BE DEEMED AS VOID IF FAULT IS FOUND TO HAVE BEEN CAUSED BY MALTREATMENT, MISUSE, ACCIDENTAL DAMAGE, INCORRECT STORAGE OR USE OF THE PRODUCTS FOR OPERATIONS OUTSIDE THEIR SPECIFIED LIMITATIONS OR OUTSIDE THEIR SPECIFICATIONS, CONTRARY TO THE INSTRUCTIONS GIVEN IN THE INSTRUCTION MANUAL.

The period of this warranty for the Distributor is defined in the instruction manual of the Products and will commence from the date the relevant Product is shipped by the Manufacturer. In case of interpretation disputes the English text applies.

All Biohit diagnostic kits have been manufactured according to our ISO 9001 / ISO 13485 quality management protocols and have passed all relevant Quality Assurance procedures related to these products.

In case of any serious incident in relation to the product, contact the manufacturer.

16. ORDERING INFORMATION

Pepsinogen II ELISA test kit.
Cat. No. 601020.02.



BIOHIT OYJ

Laippatie 1
00880 Helsinki, Finland
Tel: +358-9-773 861
E-mail: info@biohit.fi
www.biohithealthcare.com

17. EXPLANATION OF THE SYMBOLS USED IN LABELS

	CE Marking
	For <i>in vitro</i> diagnostic use
	Catalogue number
	Batch code
	Use by
	Consult instructions for use
	Temperature limitation. Store at +2...8°C
	96 determinations
	Do not re-use
	Manufacturer

18. SHORT OUTLINE OF THE PROCEDURE

Allow all the reagents to reach room temperature (20-25°C)
Remember to mix all the reagents and samples well just before
pipetting

*

After mixing, pipette 100 µl of the blank solution, the calibrators, the
control and diluted (1 to 5) patient samples into the wells

*

Incubate for 60 min at room temperature (20-25°C)

*

Wash 3 times with 350 µl of the diluted washing buffer

*

Pipette 100 µl of the mixed conjugate solution into the wells

*

Incubate for 60 min at room temperature (20-25°C)

*

Wash 3 times with 350 µl of the diluted washing buffer

*

Pipette 100 µl of the mixed substrate solution into the wells

*

Incubate for 30 min at room temperature (20-25°C)

*

Pipette 100 µl of the mixed stop solution into the wells

*

Read at 450 nm within 30 minutes