



# BIOHIT Helicobacter Pylori UFT300 quick test

## Ordering details:

### BIOHIT Helicobacter pylori UFT300 quick test with plate

REF	Item
602005PLA	BIOHIT Helicobacter pylori UFT300, 5 tests
602019PLA	BIOHIT Helicobacter pylori UFT300, 50 tests

### BIOHIT Helicobacter pylori UFT300 quick test with tube

REF	Item
602019	BIOHIT Helicobacter pylori UFT300, 50 tests
602021	BIOHIT Helicobacter pylori UFT300, 100 tests

### REF Item

602017	Positive control reagent
--------	--------------------------

## CONTACT

Biohit Oyj  
Laippatie 1  
00880 Helsinki  
Finland  
Tel. +358 9 773 861  
info@biohit.fi

Biohit HealthCare Srl  
Via Boncompagni, 3  
20139 Milano  
Italy  
Tel +39 02 38238113  
Fax +39 02 38236521  
info.italy@biohit.fi

Biohit HealthCare Ltd.  
Pioneer House, North Rd  
Ellesmere Port, CH65 1AD,  
United Kingdom  
Tel. +44 151 550 4 550  
Fax. +44 151 550 4 551  
info@biohithealthcare.co.uk


[biohithealthcare.com](http://biohithealthcare.com)

**BIOHIT**  
Innovating for Health

# BIOHIT Helicobacter Pylori UFT300 quick test



ULTRA-FAST, READY-TO-USE TEST PLATE AND TUBE VERSION

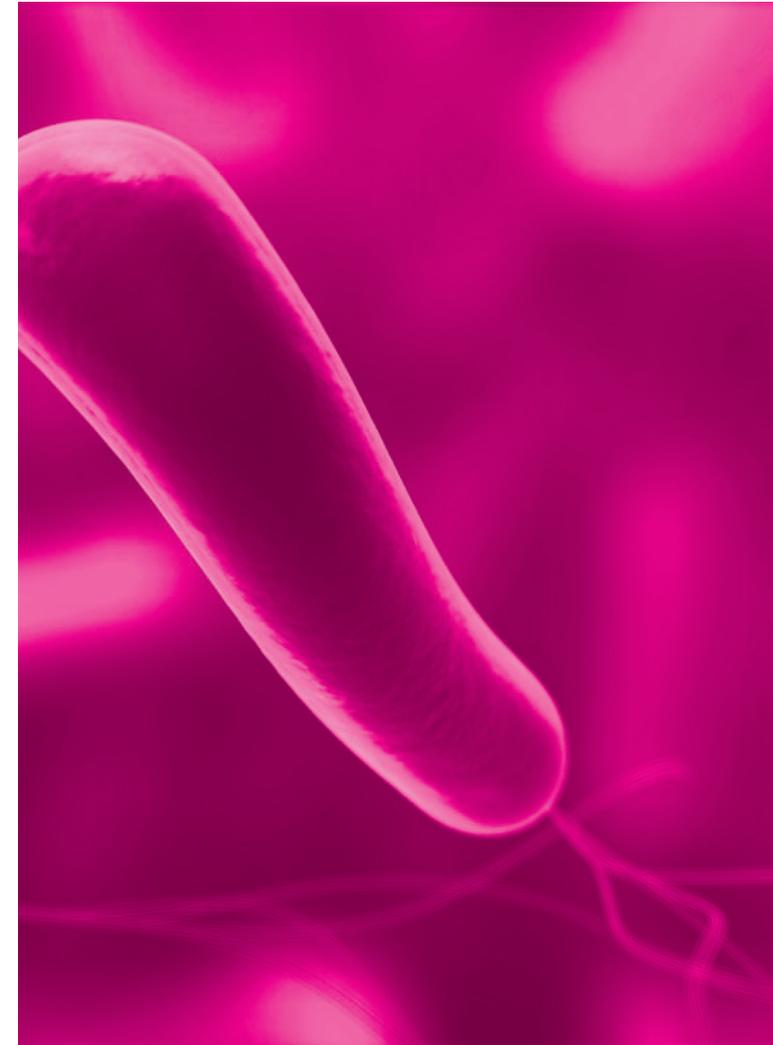

## Product specifications

- Sample: gastric biopsy specimen
- Test principle: *H. pylori* urease activity
- Kit components: test tube/plate and reagent, instruction for use
- Storage: 18-25 °C
- Quality control: positive control available



Not available in the United States. In Japan for research use only.  
Check availability for other markets.

## References

1. Vaira D, Vakil N, Gatta L, Ricci C, Perna F, Saracino I, Fiorini G, Holton J. Accuracy of a new ultrafast rapid urease test to diagnose *Helicobacter pylori* infection in 1000 consecutive dyspeptic patients. *Aliment Pharmacol Ther* 31 (2010), 331-338.

490093en-05 © BIOHIT OYJ 08/2023


**BIOHIT**  
Innovating for Health

# BIOHIT Helicobacter Pylori UFT300 quick test


The result is ready in only 5 minutes.

## Simple test procedure with plate

1. Place 1 or more biopsies into the well of the plate.

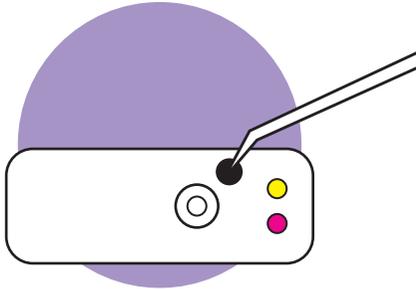


2. Add one drop of the reagent solution. Cover the plate with label and shake for 5 seconds.

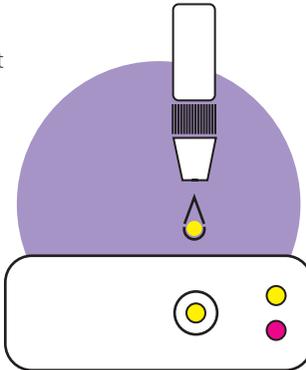


3. Read after 5 minutes.

Negative

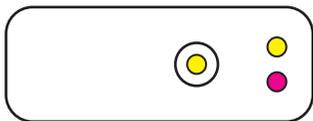

Positive

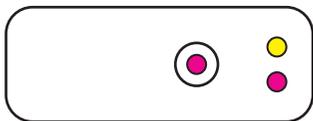

## Ultra-fast *H. pylori* detection from a biopsy

The BIOHIT *H. pylori* UFT300 is a true quick test for the detection of *H. pylori* from a biopsy. The biopsy taken during gastroscopy can be tested immediately to diagnose *H. pylori* infection or to check the success of eradication therapy. The result is ready in only 5 minutes enabling diagnosis and reporting during the same visit to the doctor. The UFT300 quick test has excellent sensitivity and specificity which makes it a highly reliable tool for diagnostics.


- Ready-to-use test kit
- Results ready in 5 min
- Storage in room temperature
- Sensitivity 94,5 %, Specificity 100 %<sup>1</sup>
- Testing and reporting during one appointment

## Biopsy testing could not be easier


The BIOHIT *H. pylori* UFT300 quick test is easy and effortless to use. The biopsy specimen is placed into the test tube/well of the plate and mixed with the test reagent. Color change indicates the presence of *H. pylori* in the specimen. The procedure is safe and hygienic for the user, and the interpretation of the result doesn't require specialist training.

## Simple test procedure with tube

1. Place 1 or more biopsies into the tube.

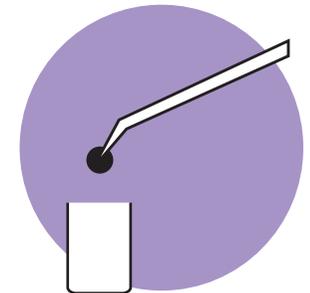

2. Add one drop of the reagent solution. Cap the tube and shake for 5 seconds.

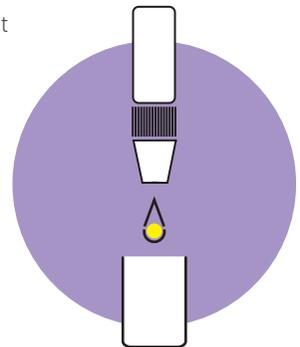

3. Read after 5 minutes.



Negative



Positive

