

Product specifications



- Sample: fingerprick blood, plasma or serum
- Test principle: immunological detection of anti-tTG antibodies (IgA, IgG and IgM)
- Kit components: lancet, micropipette, test cassette, buffer, instruction for use
- Storage: 2-8°C



Not available in the United States. In Japan for research use only.
Check availability for other markets.

BIOHIT Celiac quick test



Ordering details:

REF	Product
602070	BIOHIT Celiac quick test, 20 tests

CONTACT

Biohit Oyj
Laippatie 1
00880 Helsinki
Finland
Tel. +358 9 773 861
info@biohit.fi

Biohit HealthCare Srl.
Via Figino 20/22,
20016 Pero (Milano)
Italy
Tel +39 02 38238113
Fax +39 02 38236521
info.italy@biohit.fi

Biohit HealthCare Ltd.
Pioneer House, North Rd
Ellesmere Port, CH65 1AD,
United Kingdom
Tel. +44 151 550 4 550
Fax. +44 151 550 4 551
info@biohithealthcare.co.uk

www.biohithealthcare.com

BIOHIT
Innovating for Health

BIOHIT Celiac quick test



RELIABLE FIRST-LINE DIAGNOSTICS



490068en-05 © BIOHIT OYJ 05/2024

BIOHIT
Innovating for Health

BIOHIT Celiac quick test

Accurate results from fingerprick blood

BIOHIT Celiac quick test enables accurate and patient-friendly testing of celiac disease from only a drop of blood. The test is based on the detection of anti-tissue transglutaminase antibodies (tTG); (e.g. IgA, IgG, IgM), which delivers superior sensitivity, especially in patients with IgA deficiency.

- Results in 10 minutes
- Fingerprick sample collection
- Easy-to-use lateral flow quick test

Easy testing at the Point-of-Care



The blood sample for BIOHIT Celiac quick test is taken as a fingerprick sample. The sample is placed on the test cassette along with a buffer solution. The test result is clearly visible within 10 minutes.

Cost-effective,
proven performance

Easy sampling - fast results



1. Collect the fingerprick sample.

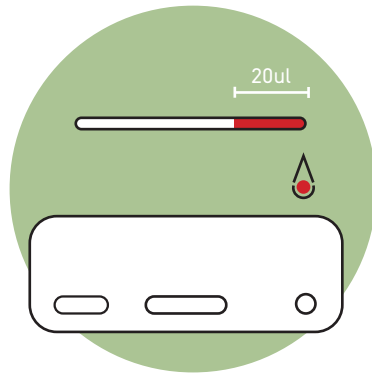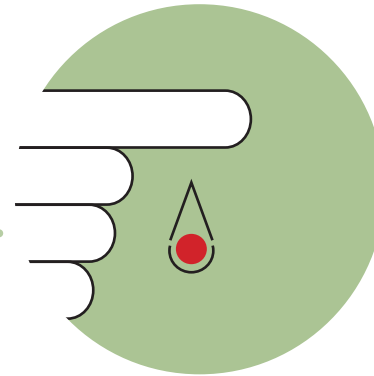


2. Dispense blood sample (20 ul) and two drops of buffer on the test cassette.




Convenient, even for pediatric patients

NEGATIVE

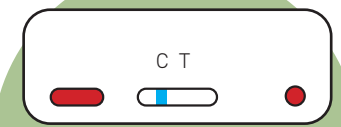

POSITIVE

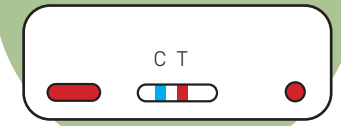

3. Read after 10 minutes