

VERSATILE TOOL FOR COLORECTAL CANCER DETECTION



Detects bleeding derived from lower GI tract

Detection of occult blood in feces is essential in the early diagnosis of colorectal cancer. The BIOHIT ColonView® quick test is an immunological lateral flow test which detects hemoglobin and hemoglobin-haptoglobin complex (degraded hemoglobin) from stool. With this combination of two biomarkers, the ColonView test can be used for detecting bleeding originating in the proximal and distal colon. This makes ColonView a superior tool to detect bleeding cancer tumors and their precursors.

- Detects bleeding from proximal and distal lesions
- Adenoma & carcinoma: sensitivity 97 % and specificity 85 %¹
- Hygienic sampling, transportation and analysis
- No dietary restrictions before sample collection
- Results in 15 min

Product specifications

- Sample: stool
- Test principle: immunological detection of Hb and Hb/Hp complex
- Kit components: sampling tube, sampling paper, test cassette, instructions for use
- Storage: 2-30 °C
- Quality control: positive control available



One sample for dual testing

The clever design of the ColonView sampling tube eliminates patient-errors and ensures that there will always be the correct sample amount. The sampling tube contains all the necessary reagents and no additional accessories are needed for the analysis. One sample is enough for the testing of both Hb and Hb/Hp complex, and the result is ready in 15 minutes.

Ordering details:



REF	Product
602250.02	BIOHIT ColonView® quick test, 30 tests
602390	BIOHIT ColonView® QT Control


Testing made easy

The stool specimens are collected easily with the stool collection paper and sample applicator stick. To guarantee the best possible accuracy, it is recommend to take samples on 3 consecutive days.



1. Make bowel movement onto the collection paper.

2. Unscrew the blue end and remove the sample applicator.

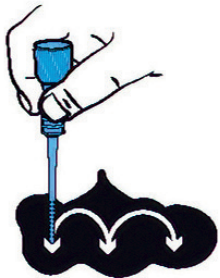


3. Collect sample by inserting the stick into the stool specimen.


4. Re-insert applicator stick.



5. Unscrew the white cap and apply drop three drops to the test cassette.

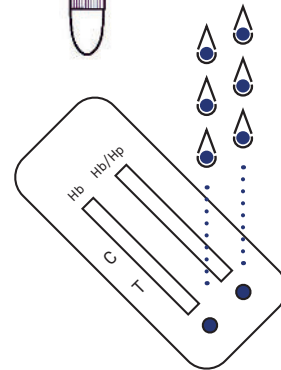

6. Read after 15 minutes

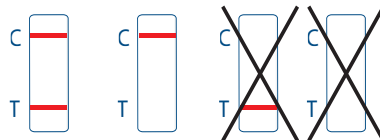

Positive Negative Invalid

CONTACT

Biohit Oyj Laippatie 1,
00880 Helsinki, Finland
Tel. +358 9 773 861
info@biohit.fi

Biohit HealthCare Srl. Via Figino 20/22,
20016 Pero (Milano), Italy
Tel +39 02 381951, Fax +39 02 38236521,
info.italy@biohit.fi

Biohit HealthCare Ltd. Pioneer House,
North Rd, Ellesmere Port, CH65 1AD,
United Kingdom
Tel. +44 151 550 4 550, Fax. +44 151 550 4 551
info@biohithealthcare.co.uk